


OLDER TRACTION UNIT MODELS

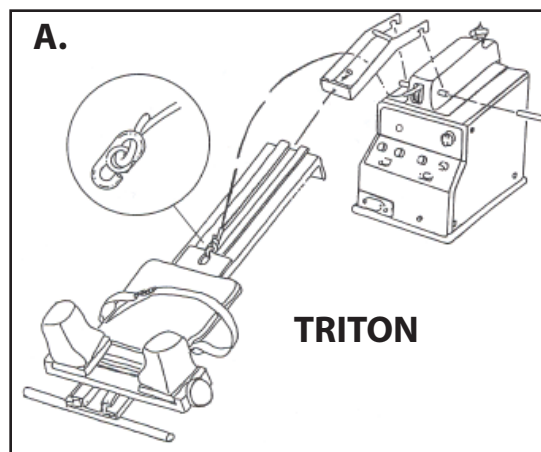TRITON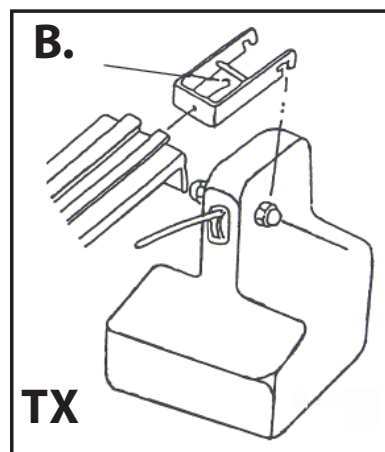

TX

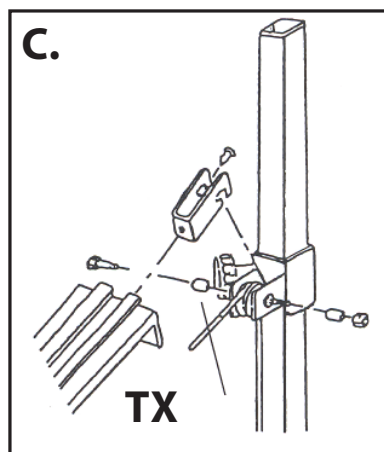TX

NEW TRACTION UNIT MODELS

ENGLISH

ASSEMBLY INSTRUCTIONS (FOR OLDER TRACTION UNIT MODELS):

A. For Triton Models:

1. Remove the existing pin from the traction unit by driving it out with the new pin.
2. Insert the new long pin, then tap back and forth to accurately center the pulley.
3. Attach the traction clevis bracket onto the long pin of the traction unit.

Attach the other end of the traction clevis to the Saunders Clinical Traction Device with a 1/4- 20 x 5/8" Socket Head Cap. Tighten to secure the clevis to the device.

4. Thread the traction rope from the unit through the traction clevis. Loop the rope onto the hook and attach the hook to the large hole at the top of the head pad.

B. For TX Models:

1. Attach the traction clevis bracket onto the traction unit. Attach the other end of the traction clevis to the Saunders Clinical Traction Device with a 1/4- 20 x 5/8" Socket Head Cap. Tighten to secure the clevis to the device.

2. Thread the traction rope from the unit through the traction clevis. Loop the rope onto the hook and attach the hook to the large hole at the top of the head pad.

C. For TX Models that use the TX Bracket:

1. Remove the screw on the upper vertical tube of the traction unit. Add the insert sleeves and replace the nuts after inserting the sleeves. Attach the TX traction clevis bracket onto the sleeves.

2. Attach the other end of the TX traction clevis to the Saunders Clinical Traction Device with a 1/4- 20 x 5/8" Socket Head Cap. Tighten to secure clevis to the device.

3. Thread the traction rope from the unit through the traction clevis. Loop the rope onto the hook and attach the hook to the large hole at the top of the head pad.

ASSEMBLY INSTRUCTIONS (FOR NEW TRACTION UNIT MODELS):

1. Attach the traction clevis bracket into the clevis attachment slots of the traction unit.

2. Attach the other end of the traction clevis to the Saunders Clinical Traction Device with a 1/4- 20 x 5/8" Socket Head Cap. Tighten to secure the clevis to the device.

3. Attach the traction rope hook to the large hole at the top of the head pad.

SETUP AND USE INSTRUCTIONS:

1. Adjust the angle of the device by raising or lowering the height of the table traction pedestal.
2. Separate neck wedges sufficiently by turning either knob on each side of the device.
3. Ask the patient to remove earrings (if any).
4. Position the patient in the device with the neck wedge located at the mid cervical region- about 1.5 in (3.81 cm) below the occipital bone.
5. Tighten the neck wedge very firmly against the neck.
6. Manually rotate the neck wedges to provide optimum fit.
7. Secure the head strap.
8. Attach the traction rope or hook to the large hole at the top of the head pad.
9. Give the patient the treatment interrupt switch and instruct how to activate.
10. Program the traction unit and initiate treatment.
11. During traction, if the patient's ears are being pushed up by the neck wedge, stop traction and return to step #4.

REMOVABLE NECK WEDGE REPLACEMENT INSTRUCTIONS:

Removal of Old Neck Wedge: Grasp inner, lower edge of the neck wedge (edge that faces the center of the unit). Pull the edge back slightly and upward, to dislodge from the lip of the aluminum post. Wiggle the neck wedge and pull upward to remove.

Replacement of New Wedge: Push the neck wedge down onto the aluminum post. Lift the bottom edge of the neck wedge and push completely down over the aluminum lip.

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© 2008 Encore Medical, L.P.